

Model: DFS-2

Manufactured and imported by Bollé Protection / Made in China

Europe (HQ) :	USA:	APAC:
Bollé Protection	Bollé Safety Inc.	Bollé Safety AU Pty Limited
34 rue de la Soie	1896 Rutherford road	P.O Box 579
69100 Villeurbanne - FRANCE	Carlsbad, CA 92008 – USA	Moorabbin VIC 3189 - Australia



This face shield product has been assessed as compliant as a PPE class 2 according to the European regulation EU 2016/425 and to the European Commission's 2020/403 document of March 2020 in the context of the COVID-19 pandemic.

All declarations of conformity are available online at : <http://www.bolle-safety.com/page/documentation>

FIELD OF USE AND PERFORMANCE

These face shield products have been tested compliant according to the standard EN166:2001*, compliant for protection against "droplets and splashes of liquids" according to the standards ANSI Z87.1:2015 "Personal eye and face protective devices" and AS/NZS 1337.1: 2010 : "Personal Eye-Protection" and compliant for the impacts "drop ball test" according to the standards ANSI Z87.1:2015 "Personal eye and face protective devices".

*According technical testing specification for COVID-19 face-shields for use by healthcare professionals - BSI's PPE for Healthcare Professionals 2020/403 – Eye protection Technical Specification. Product does not bear EN166 markings.

This protective faceshield is manufactured for COVID-protection only. It is not a PPE device for general use and shall not be used for purposes other than protection against COVID-19

DEADLINE OF OBSOLESCENCE

This product shall not be used for more than one shift.

STORAGE:

All products must be kept in a clean, dry place, out of direct sunlight and away from abrasives, solvents or solvents vapors. Storage temperatures must be between 5 to 35°C.

CAUTION

The above safety face shield products are not meant to protect against impacts. Clean and inspect regularly. Pitted or scratched lenses reduce vision - replace immediately.

If in doubt consult your safety supervisor or the supplier.

Materials which may come into contact with the wearer's skin could cause allergic reaction to susceptible individuals. Make sure you have an adequate and secure fit at all times.

TRACEABILITY:

Traceability is done on the packaging with the PO information.

NAME AND ADDRESS OF THE LABORATORY:	NAME AND ADDRESS OF THE NOTIFIED BODY (2797):
CERTOTTICA SCRL Ufficio Commerciale ZI VILLANOVA 32013 LONGARONE BL - ITALY	BSI Group The Netherlands B.V, Say Building, John M. Keynesplein 9, 1066 EP, Amsterdam, Netherlands.